HUMANTECHNIK

**LPU-1**

**D**

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Seite 2

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**GB**

**Operation Instructions**

Page 8

Induction Loop Receiver

**F**

**Mode d'emploi**

Page 14

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**NL**

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Pagina 20

Ringleidingontvanger

**I**

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Pagina 26

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**E**

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Página 32

Receptor en bucle

1

**LPU-1**  
**Ringschleifenempfänger**

### **Standard-Lieferumfang**

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**LPU-1**  
**Induction Loop Receiver**

### **Standard components**

- *LPU-1* induction loop receiver with 1 rechargeable battery
- Transparent cover
- Operating instructions
- Warranty card

### **Special accessories**

- A-4977-0 Single charger for receiver
- A-4976-0 Multiple charger for 5 DIR receivers
- A-4974-0 Multiple charger for 12 accus
- A-4955-0 Plastic shell for 5 stethoscope receivers *LPU-1*

### **Operating principle**

The induction loop receiver can receive signals which are transmitted by induction loop systems. This corresponds to a hearing aid in the »T« position.

2


Before first use, the batteries have to be charged for at least 14 hours. Only use the original batteries.

3

Make sure that the stethoscope receiver is worn correctly.

4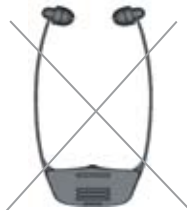


### Reception range

Ensure that the induction loop receiver is operated within an installed induction loop. Outside of this area, the induction loop receiver operates with limited capacity only.

**5**

max. ← → min.

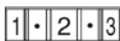**Volume adjustment of the receiver****There is no ON/OFF switch.**

The receiver automatically switches on when worn.


**6**

If the sound quality of the receiver becomes noisy and distorted the battery should be exchanged by a loaded one.

**Warning: do not force the battery into the charger or receiver. It fits easily when inserted correctly.**

**How to get the most out of your rechargeable batteries**

Please make sure that the batteries are used on a regular basis. Constantly recharging of the batteries will shorten their service life.

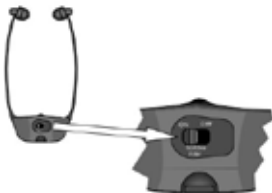**7****50/60 Hz filter**

The 50/60 Hz filter is used to suppress any annoying noise or buzzing in the induction loop receiver.

Use the ON/OFF switch on the back of the device to switch the 50/60 Hz filter on or off.

**8****Cleaning the hygienic covers**


The soft silicone earpieces should always be kept clean.

If they need to be replaced, please use the silicone earpieces available as a special accessory for DIR receivers (Order No. A-4985-0).

**9****Using the transparent cover of the volume control:**


Remove the silicone cover and replace it with the transparent cover.

Transparent cover for the volume control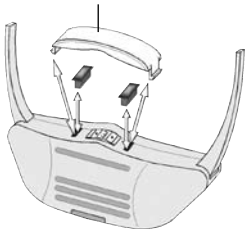

The transparent cover has the following protective functions:

- a) It prevents any unintentional changes in the volume level,
- b) it protects dust and liquids from coming into contact with the volume control.

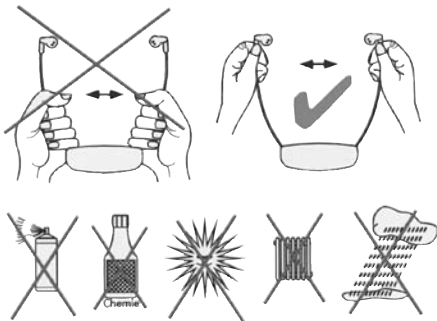**10****Please note the following information:**

If the system needs cleaning **ONLY** use a soft damp cloth. If the ear pieces become damaged they should be replaced.

The product warranty does not cover normal wear or damage resulting from misuse, neglect, improper storage, accident, unauthorised repair or alteration.

### Important Notice

This product is not protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

11

#### Warranty induction loop receiver

The *LPU-1* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly. This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product number in any event.**

12

**Disposal** of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system). The symbol on the product  or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

**13****Technical data LPU-1**

Frequency response	70 - 5400 Hz
Distortion	< 1 %
Noise level	typ. 60 dB
Battery charging time	approx. 14 h
Operating time	approx. 8 h
Maximum volume SPL	approx. 120 dBA
Weight	55 g

This product complies with European Union norms.  
You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

Technical specifications subject to change without notice.

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**Humantechnik GHL AG**